


Below is a summary of steps to follow each time a Limited-Use Public Health Emergency Verasuit™ encapsulating coverall is worn. See www.verasuit.com/donning for more details.

1. For professional health care worker use only.
2. Inspect all parts before use.
3. Only use with a verified VeraSuit™ gown, shown here.
4. Hardware needed: helmet, battery, fan, and cooling undershirt with spacers.
5. See below for cooling undershirt sizing guide.
6. Confirm that the battery is fully charged with all five charge lights illuminated. Never use a battery with less than all five lights charged. The battery lasts about eight hours with a full charge. A fully closed coverall is a suffocation risk without a working fan.
7. Find a donning partner. Donning requires a partner.
8. Wash hands before donning.
9. See www.verasuit.com/donning for the donning video.
10. See the reverse side for donning steps.
11. Before use, it is mandatory to practice donning and doffing the apparel at least two to three times, or until the wearer is comfortable with donning and doffing the apparel.

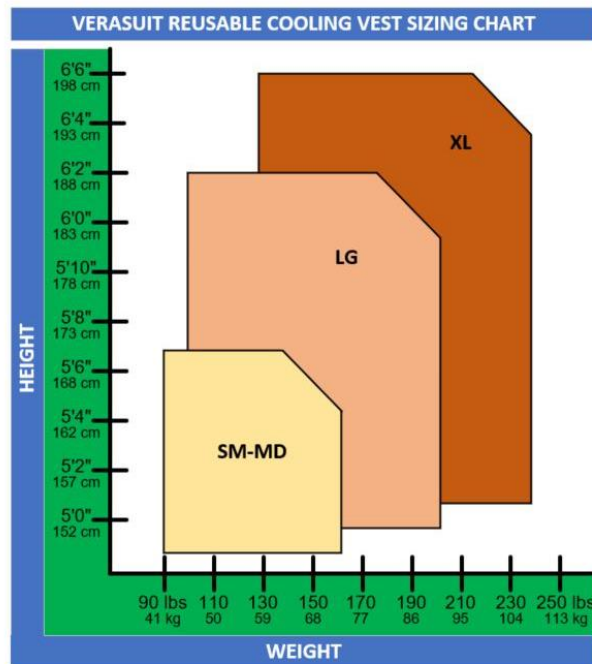

⚠ WARNING

Before using Verasuit's encapsulating coveralls, the wearer must read and understand the *User Instructions*. This product, when used as part of a complete protection system, helps protect against certain airborne and liquid contaminants. Follow all local regulations. In the U.S., a written protection program must be implemented meeting all the requirements of OSHA 29 CFR 1910.1030 and OSHA 29 CFR 1910.132, including training, fit testing, and medical evaluation. For correct use, consult your supervisor and the *User Instructions* at www.verasuit.com. **Misuse may result in serious bodily injury, sickness, or death.**